



Do. Tobell S.r.l.

Produzione parafarmaceutici

Via dell'Artigianato, 32 - 36030 SARCEDO (VI) Italy
Tel. ++39/0445.366844 - Fax ++39/0445.378966
R.E.A. VI 142980 - Export V1027495 - C.S. € 30.000 i.v.
Registro Imprese VI/Cod. Fisc./Partita IVA 00628490245
http://www.do-tobell.com - e-mail: info@do-tobell.com



SISTEMA DI QUALITÀ CERTIFICATO
Norma ISO 9001:2000 Norma ISO 13488:1996

UREA TREATMENT AGAINST MYCOSIS

Recommended plaster for topical treatment of mycosis of toenails and fingernails. The 40% of Urea contained in the plaster acts by modifying the pH of the area subject to treatment, weakening the part of the diseased nail with the consequent selective detachment of such part and facilitating its cutting and, therefore, its removal.

An improvement in the nail status will be noticed; the defect will stop and the nail will grow back healthy moving forward the damaged part that has to be progressively eliminated with the regular nail cut.

It is essential to start the treatment right when the first signs of mycosis are detected, in order to obtain more rapid and effective results.


Our Product consists of:

xx pcs. Disks with Urea

xx pcs. Occlusive fixing plasters

Instructions for Use:

Have the affected part take a bath in warm water for about 10 minutes. Dry it carefully. Remove a disk from its transparent support and apply it directly on the nail. If the disc is larger than the mycotic nail, reduce it to the right size so as to cover only the surface to be treated. The disk can be easily cut. Use the occlusive plaster to fix and protect the disk. It is advisable to replace the disc and the plaster every day for a treatment ranging from 1 to 3 weeks and in any case until the mycosis is completely removed.

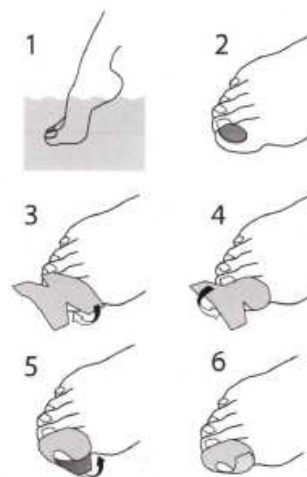

Warnings:

Do not use for periods longer than those indicated. Long treatments could cause sensitization phenomena; in this case, discontinue the treatment. Do not apply on the mucous membranes and on the skin. Do not apply on injuries. Do not use if you are allergic to one of the components. Keep out of reach of children. External use. Do not reuse.

Classification: class I Medical Device



Symbols to be used:



XXXXXXXXXX



XXXXXXX



XXXX-XX



DO. TOBELL S.r.l. Via dell'artigianato, 32 – 36030 SARCEDO (VI) - Italy